NL - DerMel Huidzalf

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EN - DerMel Skin ointment

Instructions for use

Product
DerMel Skin ointment
DerMel Skin ointment (tube 15 g, 50 g) can be used on superficial wounds such as patches of eczema.

Ingredients
Arachis oleum, cera alba, glyceryl oleate, mel, aqua, rosa oleum.

Description of product
DerMel Skin ointment is a soothing ointment that protects and heals irritated, dry and/or sensitive skin caused, for example, by eczema. DerMel Skin ointment can also be used to treat superficial and even infected wounds such as those caused by scratching patches of eczema. DerMel Skin ointment consists of a skin-care ointment combined with pure, enzyme-rich honey. The ointment's special composition, which gives it a low pH value, reinforces the skin's natural resistance. DerMel Skin ointment has a gentle cooling effect on the skin and therefore relieves itchiness.

Indication

DerMel Skin ointment can be used to care for, protect and heal irritated, dry and/or sensitive skin caused, for example, by eczema. DerMel Skin ointment

is also suitable for treating superficial and infected wounds such as those caused by scratching patches of eczema, chapped skin, and grazes. DerMel Skin ointment is suitable for babies, children and adults.

How to apply

- Unscrew the white cap from the tube.
- Turn the cap over.
- Open the tube by pressing the tip of the cap through the tube opening.
- Gently massage a thin film of ointment into the affected skin several times a day.

Use during pregnancy and while breast-feeding

There are no indications that DerMel Skin ointment should not be used during pregnancy or while breast-feeding.

When should DerMel Skin ointment not be used?

There are no indications that DerMel Skin ointment is known to be allergic or hypersensitive to any of the ingredients. An allergic reaction or hypersensitivity can cause redness, irritation and/or soreness of the skin or wound.

Side effects

- The product may cause a slight tingling sensation for a few minutes after application, probably due to the low pH value and the high osmolarity of the honey. It does not affect the effectiveness of the product.
- Please contact your doctor or the manufacturer if any unexpected side effects occur. Side effects may cause the patient discomfort.

Precautions

- Do not use if the tube is damaged as the product could become contaminated with bacteria which, in turn, could cause wound infection. Thanks to the antibacterial protection honey offers, the risk of wound infection is considered very low.
- Always replace the cap onto the tube after use. An open tube could result in bacterial contamination and wound infection. Thanks to the antibacterial protection honey offers, the risk of wound infection is considered very low.
- Avoid direct contact between the tube and the wound surface. Direct contact between the wound/skin and the (opened) tube could result in bacterial contamination and wound infection. Thanks to the antibacterial protection honey offers, the risk of wound infection is considered very low.
- Do not apply this product for more than 30 consecutive days. Treatment for more than 30 consecutive days could cause the patient discomfort such as irritation and/or soreness.
- The contents of the tube can be used for up to 3 months after opening. Longer use could reduce the effectiveness and result in contamination of the product. Thanks to the antibacterial protection honey offers, the risk of wound infection is considered very low.
- Do not use the same tube to treat different patients. This could result in cross-contamination with bacteria and wound infection. Thanks to the antibacterial protection honey offers, the risk of wound infection is considered very low.
- Do not use the product after the expiry date. This could reduce the effectiveness of the product and cause the patient discomfort.
- Do not use sharp objects to open the tube. This could result in fragments of the sealing membrane getting into the ointment, which could cause discomfort during use.

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Blactory Health Products B.V. Remmerden 58, NL-3911 TZ Rhenen